The Great Plains Laboratory, Inc.

11813 W. 77th Street • Lenexa, KS 66214

INSTRUCTIONS FOR DRIED BLOOD SPOTS (DBS) FOR IgG FOOD ALLERGY TESTS

PLEASE READ THOROUGHLY PRIOR TO STARTING COLLECTION

QUESTIONS?
(913) 341-8949
customerservice@GPL4U.com

1. Before You Begin Collection


1. Check the contents of your kit. Your Kit Contains:
 - 2 Lancets (one extra if needed)
 - 1 DBS Collection Card
 - 1 Alcohol Swab
 - 1 Adhesive Bandage
 - 1 Gauze Pad
 - 1 Test Request Form and Instructions
 - 1 Return Shipping Envelope
 - International Kit Items:
 - FedEx Airway Bill
 - FedEx Pak (Small)
2. Include physician or health care practitioner's approval in the space provided. A physician authorization may be attached to the requisition (US Only).

2. Complete and fill out the following forms (Print Clearly):

1. Test Requisition Form
2. DBS Collection Card – (patients name & date)
3. Other forms (US Only):
 - Insurance Forms (A or B)
 - Acknowledgement of Privacy Practices


3. Rinse Hands

1. Rinse hands in warm tap water.
2. Choose the designated finger tip.
3. Clean with the alcohol swab provided.
4. Allow the area to dry.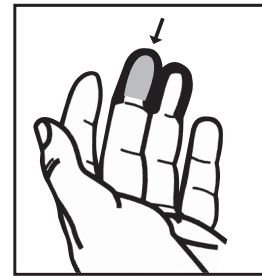

4. Stimulate Blood Flow


1. Stimulate blood flow to the selected finger by letting your hand hang at your side for 15-20 seconds.
2. Shake your hand back and forth several times.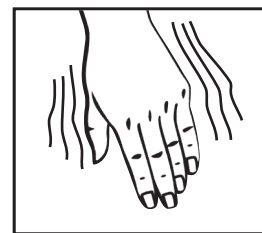

5. Use the Lancet Provided

Note: Using side of fingertip may cause less discomfort.


1. Using the lancet provided, remove the cap, place palm up and position lancet on finger.
2. Place finger firmly against hard surface and press down on Lancet to puncture site.
3. Wipe off the first blood drop with the alcohol pad.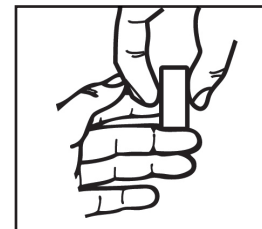


The single-use needle retracts immediately after sampling, leaving the device safe for immediate disposal in regular trash can. **DO NOT** return the lancet with sample.

6. Sample Collection


1. Blood should begin to flow freely.
2. A minimum of three blood spots (circles) are required in order to run the test. **Completely fill all five spots if possible.**
3. Place a large free-flowing drop of blood on each circle, making sure not to touch collection card with finger.
4. If blood flow stops, wipe with alcohol pad again to assist blood flow.
5. Once dried, **DO NOT** place additional sample to the same circle.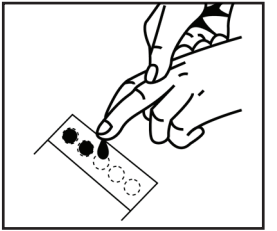

CORRECT:

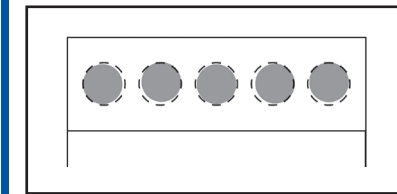INCORRECT:

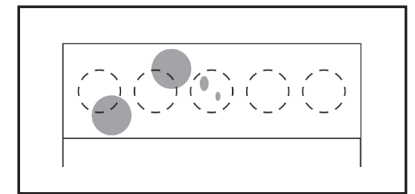7. Let The Blood Spot Air-Dry

Let the blood spots air-dry for at least 30 minutes. Make sure specimen is dry prior to mailing.

8. Mailing Instructions

Domestic:

Within 24 hours of collection, place the dried collection card into the prepaid return shipping envelope. Place completed Test Requisition Form and other forms that may apply into the envelope, seal and mail.

International:

Customer is responsible for shipping charges to The Great Plains Laboratory, Inc. The sample can be shipped via Certified Mail or via FedEx. International shipping instructions included.

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